



Swiss TPH



FRESH PREPARATION OR

DIRECT WET MOUNT

Fresh preparation or direct wet mount

Material

- Slides
- Cover slips
- Wooden sticks
- NaCl phys. (Saline)


Equipment

- Safety cabinet
- Microscope



Fresh preparation or direct wet mount

The potential infectious stool sample has to be prepared in a safety cabinet

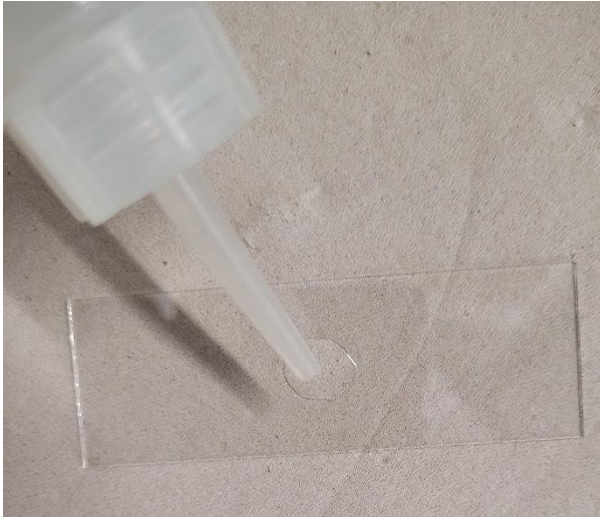

- Place a pinhead sized amount of stool in a drop of saline on the slide and mixed it well with a wooden stick. Remove coarse granular particles.

Fresh Preparation or direct wet mount



- Place cover slip and examine the preparation immediately under the microscope.
- 10x objective: Helminth eggs and larvae
- 50x objective or higher: Protozoa, Leucocytes, Erythrocytes, Macrophages